

Face shield

SPRINTIS Schenk GmbH & Co. KG
Ludwig-Weis-Str. 11 · D-97082 Würzburg
Tel. +49(0) 931 / 40 41 6-240 · Fax: -100

www.sprintis.eu · info@sprintis.eu

Storage

The face shield can be stored under normal storage conditions at room climate (approx. 50% relative air humidity & approx. 20 °C room temperature) can be stored without problems.

Usage

Before first use, the protective film applied to the outside of the face shield must be removed. The flat rubber can be stretched and pulled over the head. For the correct positioning the foam padding on the forehead place. After use, pull the face shield upwards.

Cleaning

For cleaning, moisten a cloth with water and wipe over the visible surface of the sign (inside and outside) wipe over it. Then clean the visible surface with a dry cloth.

Review

It must be checked whether there are scratches on the visible surface or whether perfect visibility is ensured despite scratches.
The face shield must be checked for cracks. If cracks are found in the face shield, the protective properties can no longer be guaranteed. The rubber band must be checked for cuts to ensure that it does not tear under load.

Disinfection

For disinfection, the face shield can be cleaned with commercially available disinfectants.

Application

The face shield should protect the wearer's face from objects.

Properties

This eye protection device (safety goggles, face shield) does not provide protection against mechanical effects.
Can be combined with face protection masks. Dispose after use. Reuse is at the discretion of the user. No accessories and spare parts available.

Risk assessment

This eye protection device does not provide protection against types of radiation (e.g. sunlight, UV, welding, IR).
This eye protection device is not fire resistant.
This eye protection device is only for use at room temperature.
Sharp edges can lead to injuries.
Do not use a damaged eye protection device, it can affect visibility and protection.
If the eye protection device is damaged, it must be replaced.

Expiry date

The expiry date of the PSA is 5 years.

Regulation

PSA Regulation (EU) 2016/425
Lenses according to EN 166: 2001
EU Declaration of Conformity (available at <https://www.sprintis.de/>)

Notified Body

MNA LABORATORIES IND.TRADE.CO.LTD.
Küçükbakkalköy Mah. Yenidoğan Cad. No.21
Ataşehir/İstanbul, Turkey
(Notified Body No.: 2841)

Labelling

SPRINTIS CE (SPRINTIS=manufacturer abbreviation, CE=EU conformity).

1.

PSA (Product, type, charge or serial number)
Gesichtsschild Artikelnr.: SIPERPLAS - 01

2.

The name and address of the manufacturer
Plasser Plastik ve Kalip San. Tic. Ltd.
Rami Kıpıla Cd. Apek Üretmen Yp Merkezi
35050 Istanbul

3.

The manufacturer bears sole responsibility for issuing this declaration of conformity:
Plasser Plastik ve Kalip San. Tic. Ltd.
Rami Kıpıla Cd. Apek Üretmen Yp Merkezi
35050 Istanbul

4.

Subject of the declaration:
Gesichtsschild SIPERPLAS - 01



5.

The subject matter of the Declaration described in point 4 is in line with the relevant Union harmonisation legislation:
PSA Verordnung 2016/425/EU

6.

EN 166: 2001

7.

Notified Body

MNA LABORATORIES IND.TRADE.CO.LTD.
Küçükbakkalköy Mah. Yenidoğan Cad. No.21
Ataşehir/İstanbul, Turkey
(Notified Body No.: 2841)

Made the EU type examination (module B) and issued the EU type examination certificate 39-20-01.